Datasheet last updated 5/9/2025 6:37:54 AM (supersedes all previous editions)

Alleima

Alleima® High-N Bar Datasheet

Alleima® High-N is a molybdenum and nitrogen alloyed austenitic stainless steel, used for orthopedic implants and fracture fixation devices.

The grade is characterized by:

- Proof strength is up to twice that of Alleima® 316LVM in the annealed condition
- Higher corrosion resistance than Alleima® 316LVM
- High microcleanliness
- Very good surface finish
- Good forgeability

Standards

UNS: S31675

BS: 7252-9

Product standard

ASTM F1586

Chemical composition (nominal)

Chemical composition (nominal) %

С	Si	Mn	Р	S	Cr	Ni	Мо	Cu	N	Nb
≤0.06	≤0.60	4.0	≤0.025	≤0.003	20.5	9.5	2.4	≤0.20	0.4	0.3

Applications

Alleima® High-N grade is used for hip and knee joints and fracture fixation devices.

Corrosion resistance

Alleima® High-N has a very good resistance in physiological environments to:

- General and intergranular corrosion due to high purity and low ferrite content
- Pitting and crevice corrosion due to the high molybdenum content

Forms of supply

Alleima® High-N is supplied as both billet and hot-rolled round bar.

Other product forms

Alleima® High-N can also be supplied as tube and wire.

Mechanical properties

Product	Condition	Tensile strength		Proof strength		Elongation A,
		R _m		$R_{p0,2}$		%
		MPa	ksi	MPa	ksi	
		min	min	min	min	typical
Wire		1500 max	217 max			
Bar	Annealed	740	107	430	62	35
Bar	Medium	1000	145	700	102	20
Bar	Hard	1100	160	1000	145	10
Tube, thick wall	Bright annealed	740	107	430	62	min 35
Tube, thick wall	Cold finished	1200	174	1100	160	min 10

Specific tensile requirements available on request

Physical properties

Property

Density (20 °C)	8.0 g/cm ³	0.29 lb/in ³
Modulus of elasticity, x10 ³ (20°C)	200 MPa	29.0ksi
Specific heat capacity (20°C)	485 J/(kg °C)	0.11Btu/(lb°F)
Thermal conductivity (20°C)	14W /(m°C)	8 Btu/(ft h °F)
Thermal expansion, x10 ⁻⁶ (30-100°C)	16.5 per °C	9.5 per °F

Machining

Hardness	Cutting speed range	Feed range
----------	---------------------	------------

		SFM	m/min	IPR	mm/rev
Turning	250-400	885-145	270-45	0.002-0.024	0.05-0.6
Milling	250-400	855-245	260-75	0.002-0.016	0.05-0.4
Drilling	250-400	115-195	35-60	0.002-0.012	0.05-0.3

Disclaimer:

Alleima is not providing any products or services that are intended or may be construed to be recommending or otherwise advising on, in any manner, the design, suitability, appropriateness or effectiveness, from a medical/biological/safety perspective, of any medical material, instrument and/or medical device.

